# Oticon ON Anleitung

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
Allow "Oticon ON" to access your location even when you are not using the app?  
The app uses location services to help you find your hearing aids.

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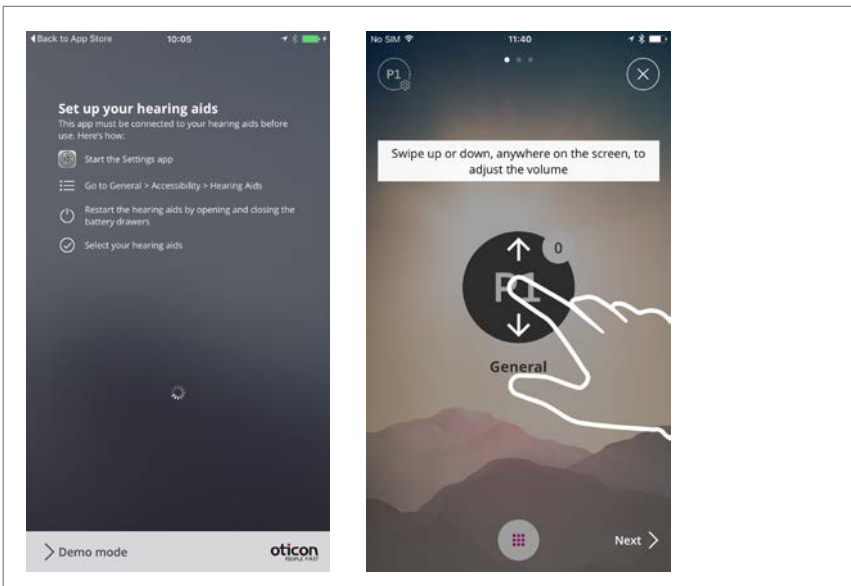
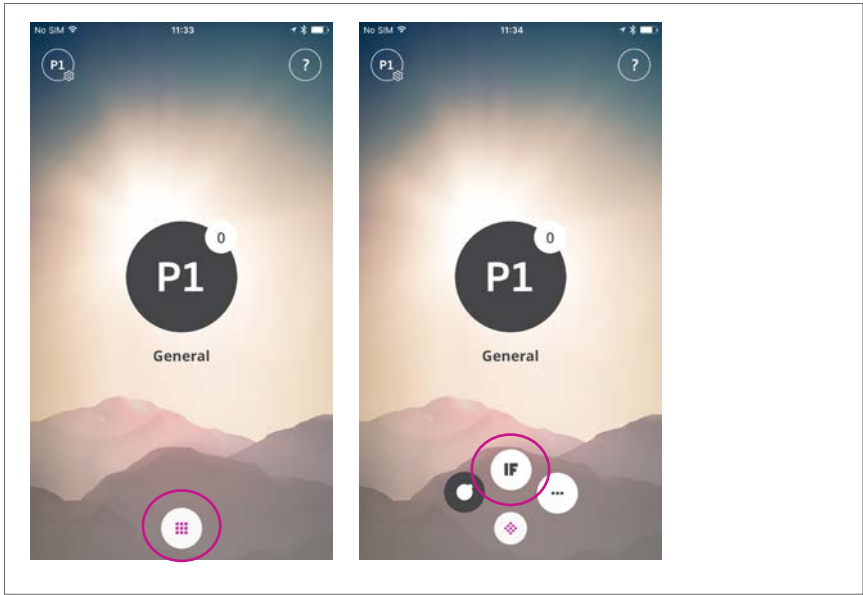
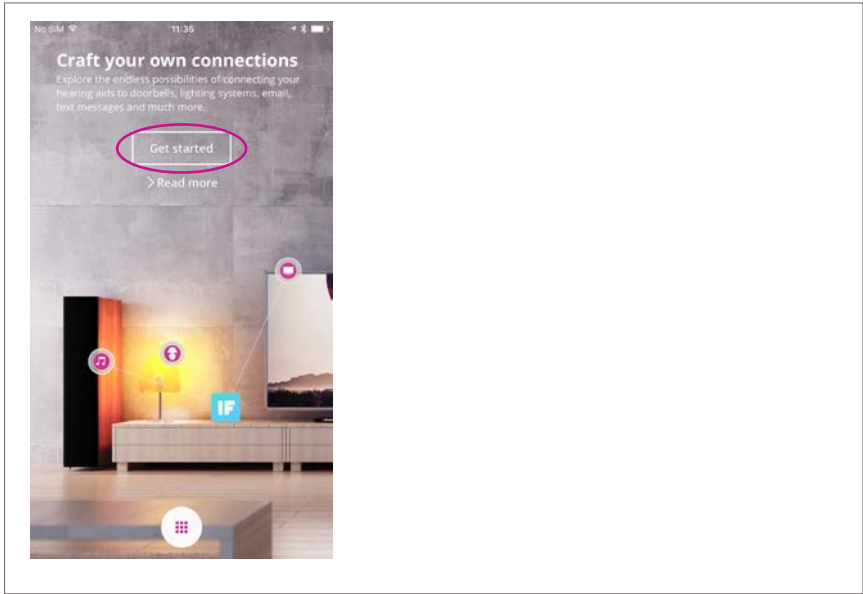
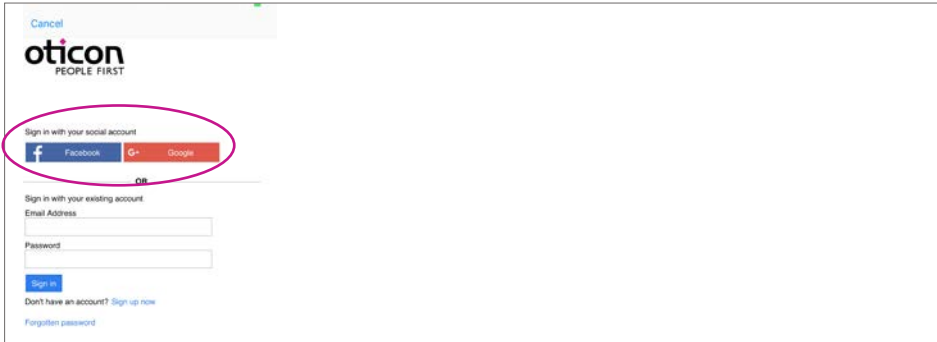
The screenshot shows the Oticon login interface. At the top, there is a 'Cancel' button and the Oticon logo with the tagline 'PEOPLE FIRST'. Below the logo, there are two options for signing in with a social account: Facebook and Google. These buttons are circled in pink. Below this, there is an 'OR' separator and a section for signing in with an existing account, which includes fields for 'Email Address' and 'Password', and a 'Sign in' button. At the bottom, there is a link for 'Don't have an account? Sign up now' and a link for 'Forgotten password'.

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
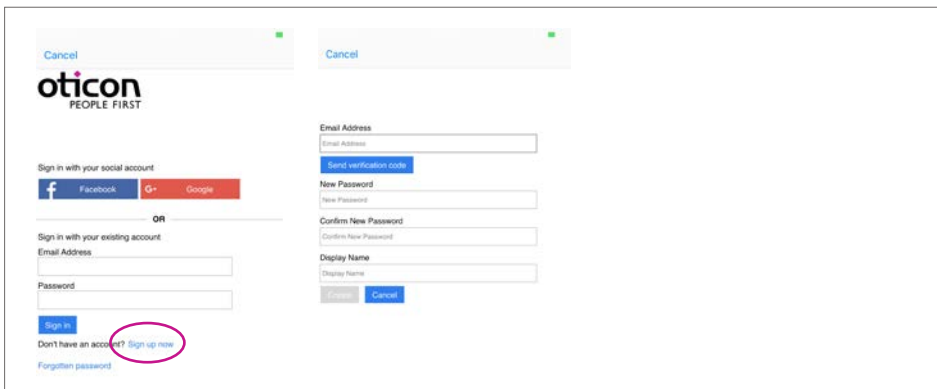
The screenshot shows the Oticon sign-up interface. On the left, there are options for signing in with a social account (Facebook and Google) and an 'OR' separator. Below this, there is a section for signing in with an existing account, which includes fields for 'Email Address' and 'Password', and a 'Sign in' button. At the bottom, there is a link for 'Don't have an account? Sign up now' circled in pink. On the right, there is a section for creating a new account, which includes fields for 'Email Address', 'New Password', 'Confirm New Password', and 'Display Name', and buttons for 'Send verification code', 'Create', and 'Cancel'.

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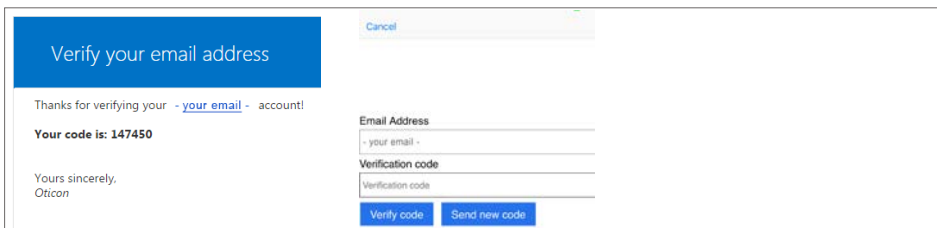The screenshot shows an email verification screen. At the top, there is a blue header with the text 'Verify your email address' and a 'Cancel' button. Below the header, there is a message: 'Thanks for verifying your - your email - account!'. Below this, there is a line of text: 'Your code is: 147450'. Below that, there is a signature: 'Yours sincerely, Oticon'. On the right, there is a form with fields for 'Email Address' (containing '- your email -'), 'Verification code' (containing 'Verification code'), and buttons for 'Verify code' and 'Send new code'.

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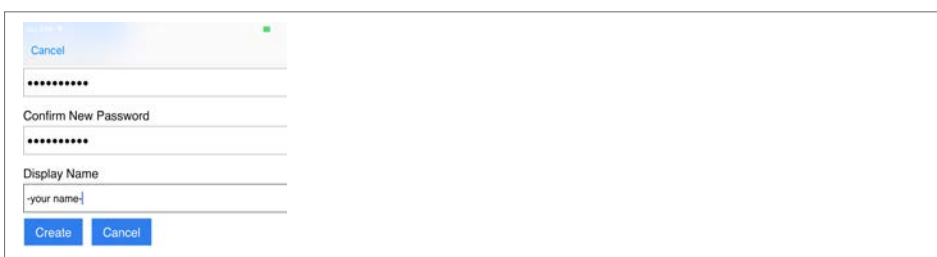The screenshot shows the Oticon account creation screen. At the top, there is a 'Cancel' button. Below it, there are three input fields: the first is for a new password (masked with dots), the second is for confirming the new password (masked with dots), and the third is for the display name (containing '-your name-'). At the bottom, there are two buttons: 'Create' and 'Cancel'.

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